



RHYTHM  
HEALTHCARE

ITEM# P2

# P2 SIEVE REPLACEMENT GUIDE



**Tools required:** T10 screwdriver X1 , tweezers X1



**Notice:**

Pls turn off the device, disconnect the power adapter before maintenance


### **Operation Procedure :**

- Remove the defective sieve bed
- Disconnect the valve base from sieve bed column
- Connect the valve base to the new sieve bed
- Install the new sieve bed to concentrator


## **Stage 1** Standby





Remove the battery by forcing down the latch ,then slide out the battery horizontally.

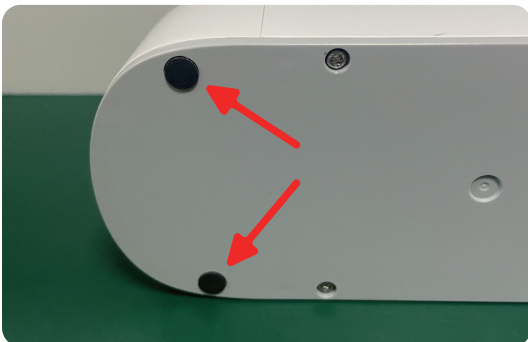

Remove the anti-slip mats installed on the screw holes on the bottom of the concentrator.

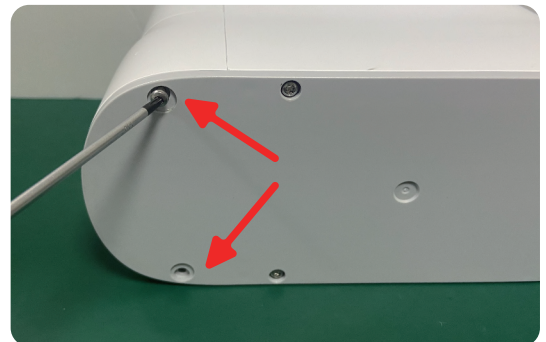


Remove the two fixing screws on the bottom of base by using the T10 screwdriver.

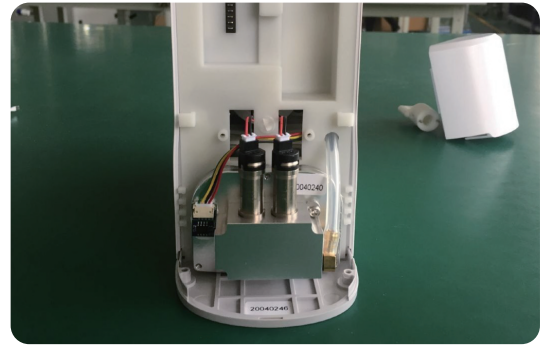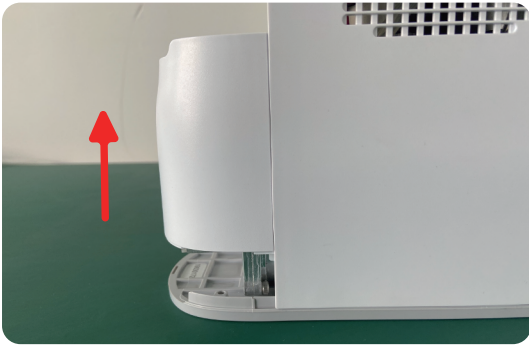


*Slide upward to remove the Sieve bed compartment cover.*


## **Stage 2** Remove the defective sieve bed

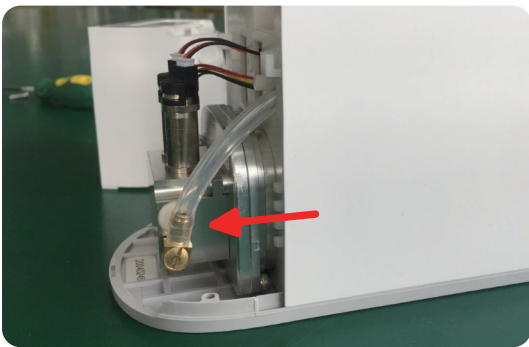


*Pull out the silicone tube by hand or tweezers.*

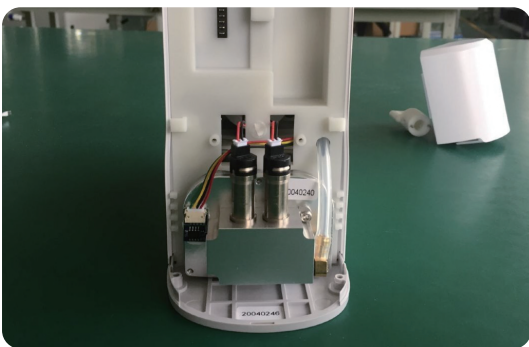

*Pull out the 3 data cables .*

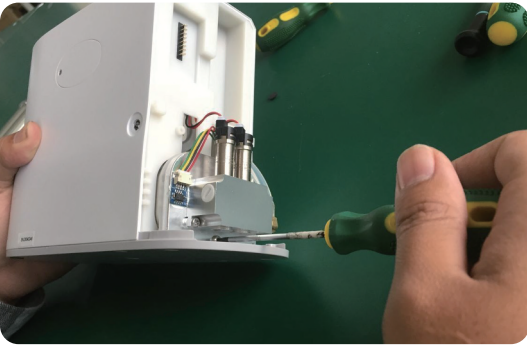

*Screw off the fixing screw on the bottom of valve base by the T10 screwdriver.*


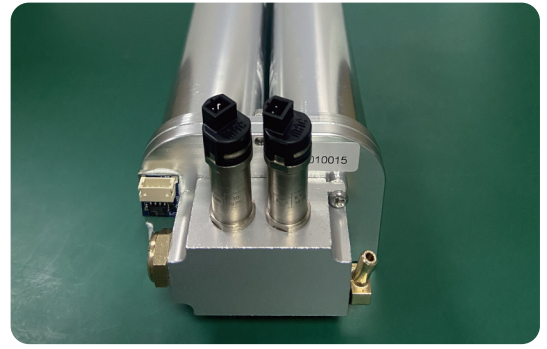


*Pull out the sieve bed assy carefully*

### **Stage 3** *Disconnect the valve base from defective sieve bed*






*screw off the two screws on valve base.*

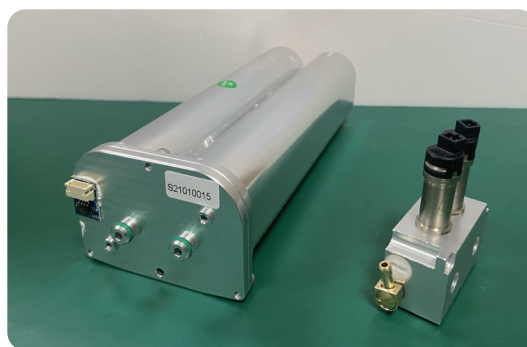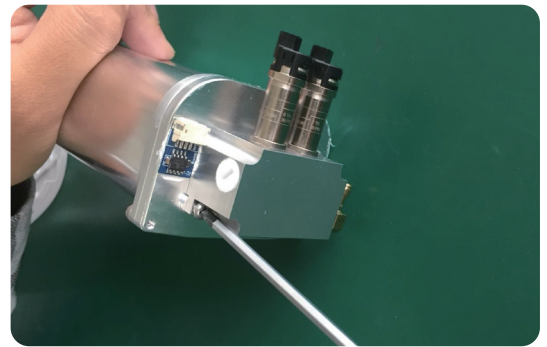

*Remove the MAC valve base from the sieve bed, take out the former sieve bed.*

## **Stage 4** Connect the valve base to new sieve bed

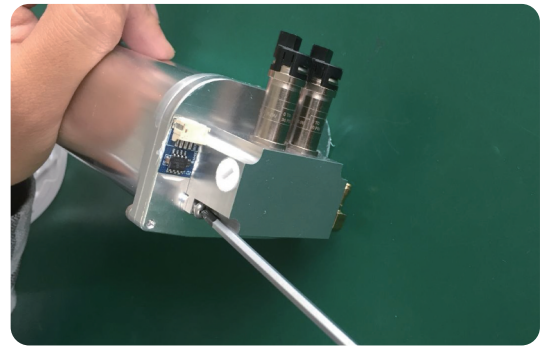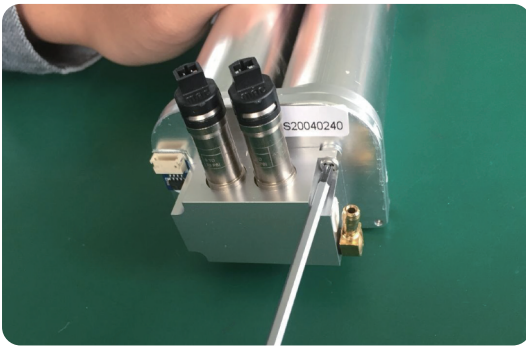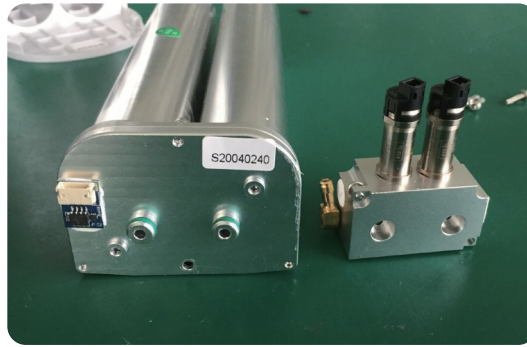


*Install a new sieve bed into the unit, then install the MAC valve on the top of sieve bed, screw the two screws back.*

## **Stage 5** Install the new sieve bed to concentrator




*Push the sieve bed assy to the end  
(please check carefully if the sieve bed  
pushed to the end)*


*Screw back the fixing screw of sieve bed  
assy.*

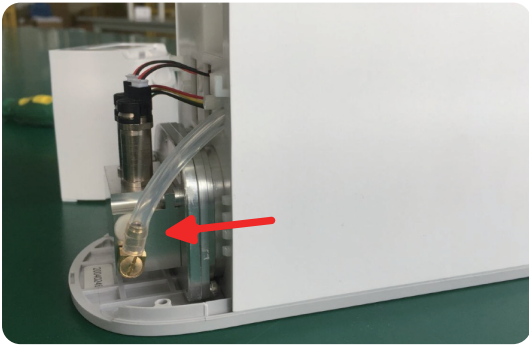


Connect the silicone tube to the joint well.

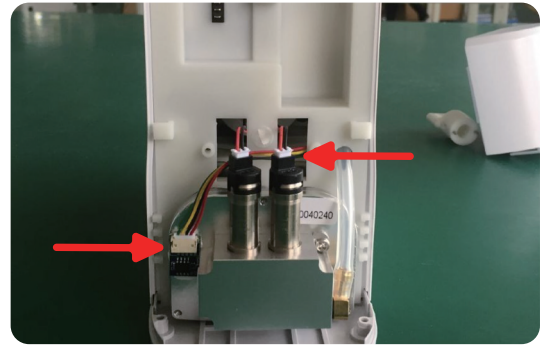


Connect the 3 data cables of the MAC valve to the valve ports.

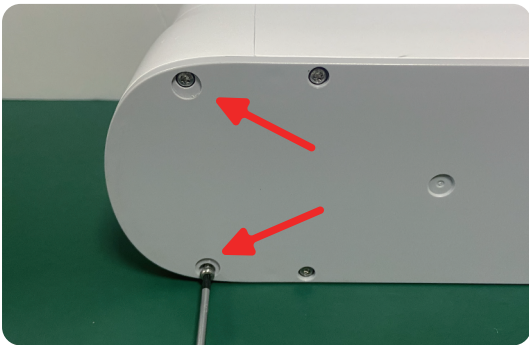

Screw on the fixing screws on the base .

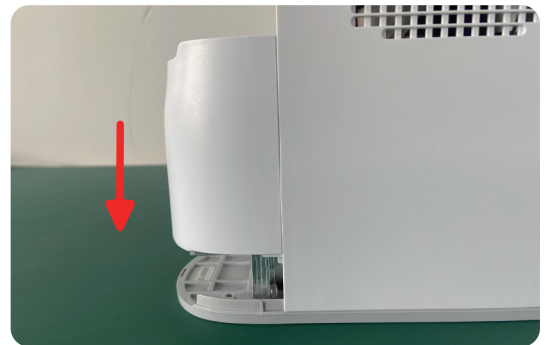

Slide down the sieve bed compartment cover to the suitable position.

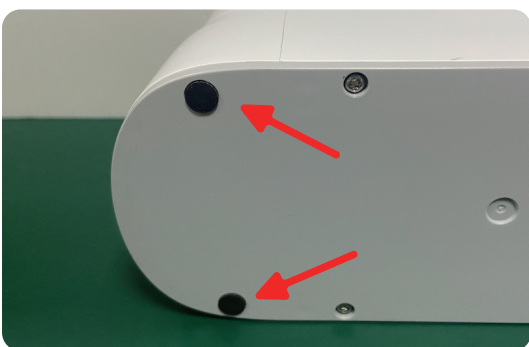

Set the anti-slip mats.

You have successfully changed the sieve bed ,then Install the battery or connect the adapter to use the oxygen concentrator .



Rhythm Healthcare, LLC  
3200 Tyrone Blvd N  
St. Petersburg, FL 33710

Email: [contactus@rhythmhc.com](mailto:contactus@rhythmhc.com)  
Website: [www.rhythmhc.com](http://www.rhythmhc.com)

V2 2/24/21